


LATERAL STABILIZER ARMREST BOLSTER

Reorder #	Description	Cover Fabric
□706110	Full-Arm/5" Diameter	Polyester
□706111	Full-Arm/5" Diameter	Synthetic Sheepskin
□706112	Full-Arm/5" Diameter	Spray-and-Wipe Nylon
□706114	Half-Arm/5" Diameter	Polyester
□706115	Half-Arm/5" Diameter	Synthetic Sheepskin
□706116	Half-Arm/5" Diameter	Spray-and-Wipe Nylon
□706120	Full-Arm/6" Diameter	Polyester
□706121	Full-Arm/6" Diameter	Synthetic Sheepskin
□706122	Full-Arm/6" Diameter	Spray-and-Wipe Nylon
□706124	Half-Arm/6" Diameter	Polyester
□706125	Half-Arm/6" Diameter	Synthetic Sheepskin
□706126	Half-Arm/6" Diameter	Spray-and-Wipe Nylon

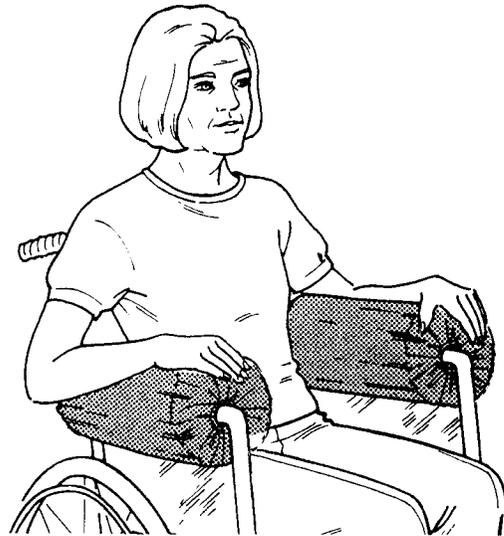

2. Remove covers and place the bolsters over the wheelchair armrests, slit side down.

3. Replace covers-open side down-and secure hook-and loop closures.

PURPOSE

The Lateral Stabilizer Armrest Bolsters are a part of SkiCare's Patient Autonomy Safety System (PASS) product line designed to reduce reliance on physical restraints. These bolsters reduce lateral leaning for wheelchair patients who lean to the left or right.

CAUTION


This device is not a restraint! It is designed to provide lateral posture support only. The determination as to the appropriateness of the Lateral Stabilizer Armrest Bolster for a specific patient is the responsibility of the facility and its professional and medical staff.

APPLICATION

1. Two bolsters will be required for patient stabilization. Select the proper size bolsters for your patient. 6-inch diameter for most patients; 5-inch diameter for heavy patients.

ALTERNATE APPLICATION

For some patients, applying the bolsters to the vertical bar on the wheelchair backrest may provide more lateral support. If you wish to use the bolsters in this manner, apply them as shown in the illustration to the right.

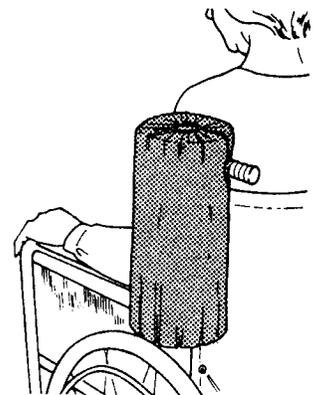

COVER LAUNDERING OR CLEANING

Both the polyester and sheepskin covers may be laundered and dried at temperatures up to 180°F (82°C) Bleaching to not recommended. To reduce the amount of lint that may accumulate in the hook-and-loop closures during laundering, we suggest that you wash and dry the covers nylon covers may be cleaned with a spray-type cleaner and a damp cloth or sponge.

Replacement covers are available from your Skil-Care dealer. See Skil-Care Price List for descriptions and reorder numbers.