



WIPESPLUS® TECHNICAL DATA

EFFICACY DATA FOR WIPESPLUS® DISINFECTING WIPES EPA REG. NO. 6836-340

PRODUCT DESCRIPTION

Disinfecting formula used to kill 99.9% of bacteria on surfaces when used as directed. Our disposable, bleach-free and highly absorbent wipes are safe to use on plastics, laminates, metals, stainless steel and glass surfaces.

Note: Do not use as a baby wipe.

CHEMICAL COMPOSITION: Virucidal*, Bactericidal, Fungicidal

ACTIVE INGREDIENTS:

Octyl decyl dimethyl ammonium chloride.....	0.0909%
Dioctyl dimethyl ammonium chloride.....	0.0364%
Didecyl dimethyl ammonium chloride.....	0.0545%
Alkyl (C14, 50%; C12, 40%; C16, 10%) dimethyl benzyl ammonium chloride.....	0.1212%
Other Ingredients:.....	99.6970%
Total.....	100.0000% ¹

USE IN:

Hospitals/Military Bases/Day Care Centers/ Nursing Homes/Schools/Universities Commercial Buildings
Food Service Area /Fitness Centers/Hotels & Resorts/Grocery Stores

FEATURES AND BENEFITS:

- More effective and efficient than sprays and towels.
- Bleach free and highly absorbent for an optimal clean.
- Independently lab certified to quickly disinfect.
- Kills Flu virus, Staph (MRSA), E. Coli, HIV-1 and 2019 Novel CoronaVirus*.
- Full spectrum EPA registered disinfectant.

*WipesPlus® Disinfecting Wipes have demonstrated effectiveness against viruses similar to 2019 Novel Coronavirus (2019-nCoV) on hard, non-porous surfaces. Therefore, WipesPlus® Disinfecting Wipes can be used against 2019 Novel Coronavirus (2019-nCoV) when used in accordance with the directions for use against Norovirus on hard, non-porous surfaces.

CONTACT TIME:

Use a 4 minute contact time for disinfection against all virus, bacteria and fungi claimed, unless otherwise stated. Use a 15 second contact time for sanitization against Staphylococcus aureus [Staph] [ATCC 6538], Klebsiella pneumoniae [ATCC 4352], and Campylobacter jejuni [ATCC 29428].

EFFECTIVE AGAINST:

Bacteria:

- | | |
|--|--|
| <ul style="list-style-type: none"> • Pseudomonas aeruginosa [Pseudomonas] • Salmonella enterica [Salmonella] • Staphylococcus aureus [Staph] • Acinetobacter baumannii • Burkholderia cepacia • Campylobacter jejuni [Campylobacter] • Corynebacterium ammoniagenes • Enterobacter aerogenes • Enterococcus faecalis [Enterococcus] | <ul style="list-style-type: none"> • Listeria monocytogenes • Salmonella schottmuelleri [Salmonella] • Salmonella typhi [Salmonella] • Serratia marcescens [Serratia] • Shigella dysenteriae [Shigella] • Staphylococcus aureus - Community Associated Methicillin-Resistant [CA-MRSA] [NRS384] [USA300] • Staphylococcus aureus - Community Associated Methicillin-Resistant [CA-MRSA] [NRS123] [USA400] |
|--|--|

- Enterococcus faecalis- Vancomycin Resistant [VRE]
- Escherichia coli [E. coli]ba
- Escherichia coli O157:H7
- ESBL Escherichia coli – [Extended spectrum beta-lactamase producing E. coli]
- Klebsiella pneumoniae [Klebsiella]
- Legionella pneumophila
- Staphylococcus aureus - Methicillin-Resistant [MRSA]
- Staphylococcus aureus - Multi-Drug Resistant [resistant to tetracycline [Tc], penicillin [Pc], streptomycin [Sm] and erythromycin [Em] and susceptible to chloramphenicol [Cm] in vitro]
- Staphylococcus aureus-Vancomycin Intermediate Resistant – [VISA]
- Streptococcus pyogenes [Strep] [a cause of scarlet fever]
- Vibrio cholerae

Viruses:

- Hepatitis B Virus [HBV]
- Hepatitis C Virus [HCV]
- Herpes Simplex Virus Type 1
- Herpes Simplex Virus Type 2
- HIV-1 [AIDS Virus]**
- Human Coronavirus
- Influenza Type A Virus / Brazil [Influenza] [[the] Flu [Virus]]
- Influenza A Virus H1N1
- 2013 Influenza A Virus (H7N9)
- Norwalk Virus, [Feline Calicivirus] [Norovirus]
- Respiratory Syncytial Virus [RSV]
- Rotavirus,
- SARS Associated Coronavirus [SARS] [cause of Severe Acute Respiratory Syndrome]
- Vaccinia Virus [Pox Virus]

Animal Viruses:

- Avian Influenza Virus H3N2
- Avian Influenza Virus H5N1
- Canine Distemper Virus
- Feline Calicivirus
- Newcastle's Disease Virus
- Pseudorabies Virus

Fungi

- Candida albicans++

**Use a 1 minute contact time

+ Treated Surfaces must remain wet for 10 minutes

++ Treated Surfaces must remain wet for 8 minutes

QUANTITY:

Item No. 37301 800ct	4 per case	8" x 6"
Item No. 33900 240ct	12 per case	6" x 6.75"
Item No. 33711 75ct	6 per case	7" x 8"
Item No. 37701 80ct	12 per case	7" x 8"

SAFETY (See product for full safety precautions):

Caution: Harmful if swallowed. Harmful if inhaled.
Avoid breathing vapors.