



A GUIDE TO STRATUS 5 FILTERS

STR1030C : Stratus 5 Sponge Filter




The Stratus 5 comes with 2 Sponge filters. This filter goes into the side of the device as shown above. This filter should be cleaned (washed in warm water and mild unscented dish soap once a week in normal operating environments. It should be replaced whenever it is worn or damaged and in between patients.

STR1020 / STR1020C: Stratus 5 HEPA Paper Filter



The Stratus 5 HEPA paper filter is located in the back of the unit as shown above. The unit comes with one HEPA filter. This filter should be changed every 10,000 hours and in between patients.

STR1040C : Stratus 5 Bacteria Filter

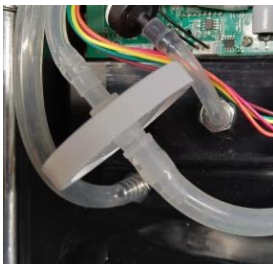

The Stratus 5 Bacteria Filter is located inside of the Stratus 5 device itself. It needs to be changed every 15,000 hours.