

NEW SOLUTIONS

OX010 INSTALLATION INSTRUCTIONS

STEP 1.

REMOVE THE ANTI-TIP.



STEP 2.

ATTACH MOUNTING PLATE TO
SHAFT WITH LARGE U-BOLT
PLACING NUT TO THE INSIDE



STEP 3.

SLIDE OX010 ONTO SHAFT.



STEP 4.

ATTACH OX010 TO BRACKET WITH SMALL U-BOLT PLACING NUTS TO THE OUTSIDE.



STEP 5.

REPLACE ANTI-TIPPER.

