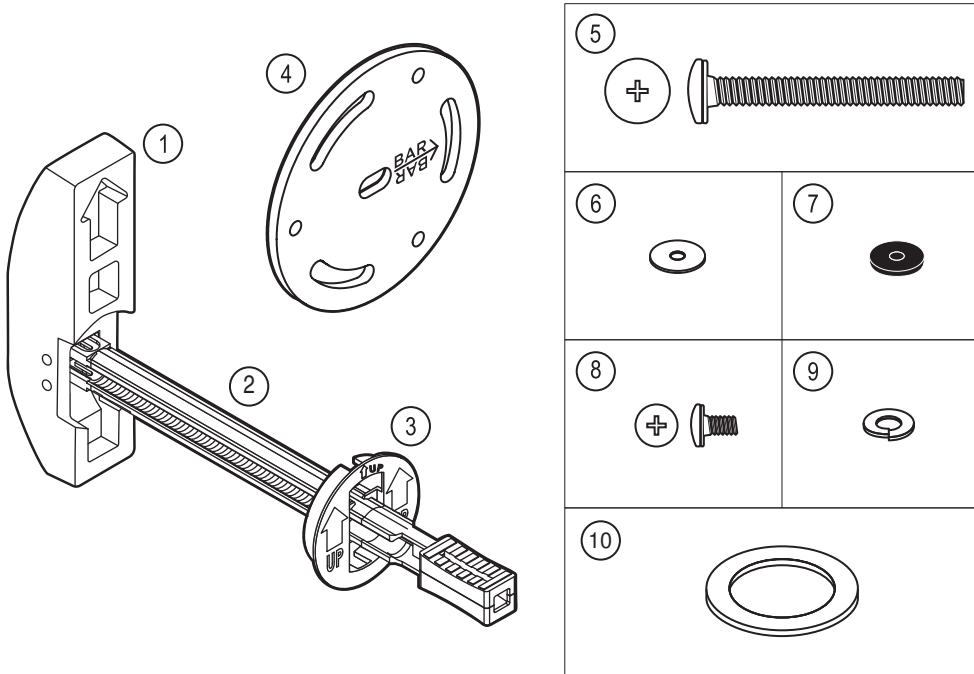
Easy Mount™ Anchor

(DC198 - RevA)

1. Anchor (x1)
Ancre (x1)
Anclaje (x1)
2. Plastic rails (x1)
Rails en plastique (x1)
Guías de plástico (x1)
3. Plastic retainer (x1)
Pièce de retenue en plastique (x1)
Retenedor de plástico (x1)
4. Mounting plate w/ foam gasket(x1)
Plaque de montage a/ joint en mousse (x1)
Placa de montaje con junta de espuma (x1)
5. M6 x 80mm machine screw, S/S (x1)
Vis mécanique M6 x 80mm, S/S (x1)
Tornillo metálico de M6 x 80mm, S/S (x1)
6. Flat washer, S/S (x1)
Rondelle plate, S/S (x1)
Arandela plana, S/S (x1)
7. Rubber washer, S/S (x1)
Rondelle en caoutchouc, S/S (x1)
Arandela de caucho, S/S (x1)
8. M5 machine screw, S/S (x3)
Vis mécanique M5, S/S (x3)
Tornillos metálicos M5, S/S (x3)
9. Split lock washer, S/S (x3)
Rondelle à ressort, S/S (x3)
Arandela de seguridad, S/S (x3)
10. Double sided tape ring, (x1)
Rondelle en ruban adhésif double, (x1)
Arandela de cinta adhesiva doble, (x1)

WALL REQUIREMENTS / EXIGENCES DU MUR / REQUISITOS DE LA PARED

Requires a 3½" space behind wall / Nécessite un espace de 3½ po sous la surface du mur / Requiere un espacio de 3½" detrás de la pared.

Drywall / Cloison sèche / Panel de Yeso: ½" or ¾"

Drywall + Tile / Cloison sèche + Carrelage / Panel de yeso + azulejo: MAX 1½"

Fiberglass / Fibre de verre / Fibra de vidrio: ⅛" or ¼"

TOOLS REQUIRED / OUTILS REQUIS / HERRAMIENTAS NECESARIAS

Drill
Perçuse
Taladro

Pencil
Crayon
Lápiz

Scissors
Ciseau
Tijeras

Level
Niveau
Nivel

Phillips Screwdriver
Tournevis Phillips
Desatornillador Philips

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
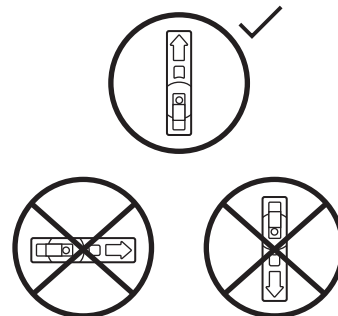
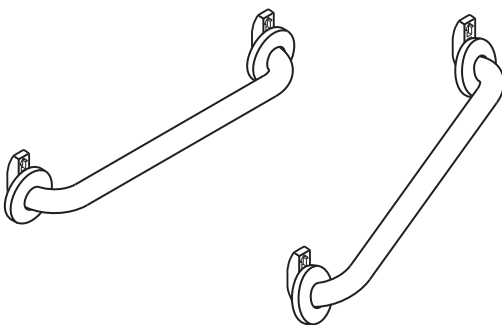
1. Must be installed on wall with space of at least 3½" behind.
2. Locate studs prior to drilling.
3. Determine location of any obstruction behind wall.
4. Ensure wall surface is even. Fill groutlines and uneven surfaces with silicone.
5. Make certain the Easy Mount Anchor is installed upright.
6. Do not modify this product.

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












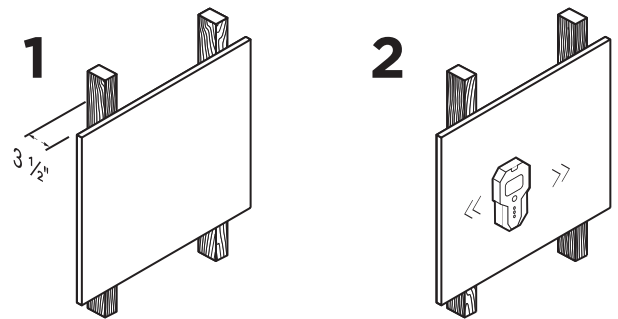
INSTALLATION / INSTALLATION / INSTALACIÓN

1. Mark outside of grab bar flanges.
2. Use mouting plate to mark center.
3. Drill hole using 1¼" hole saw. Continue to step 4 if installing on fiberglass, otherwise skip to step 8.
4. Clean the wall area around the hole.
5. Peel one side of the adhesive tape.
6. Stick the adhesive tape around the hole.
7. Peel the second side of the adhesive tape.
8. Slide top rail to pivot anchor down and fit through hole in wall. Ensure the "UP" is pointing upwards.
9. Slide top rail to pivot anchor up and pull back so anchor is flat against back of wall.
10. Slide retainer in place until tight fit against the wall.
11. Cut off and discard plastic rails.
12. Install mount plate using rubber washer, metal washer and M6 machine screw. Align the arrows on the mounting plate with the grab bar.
13. Install the grab bar using the split washers and M5 machine screws (3x).
Inspect hardware regularly, and tighten if required.

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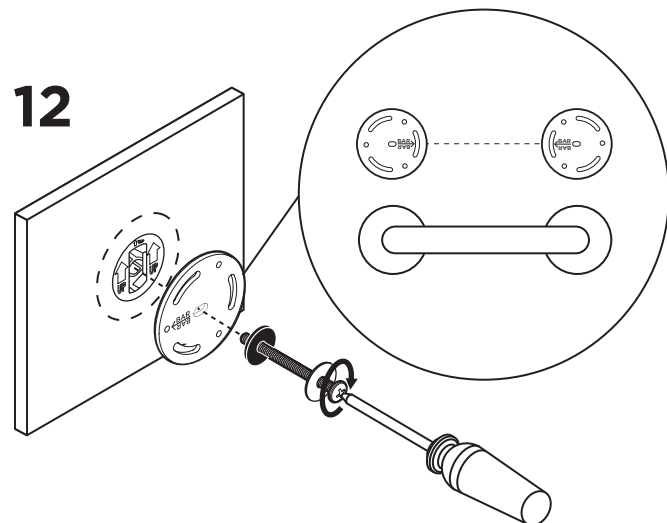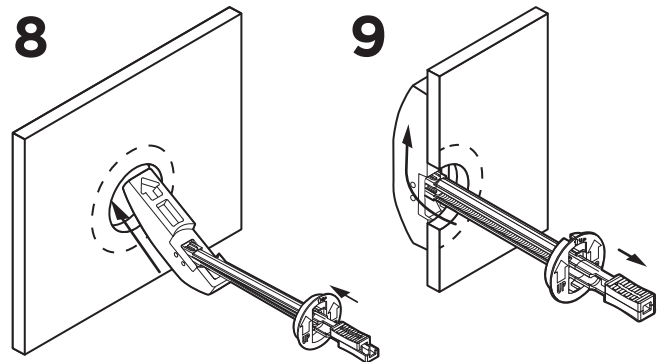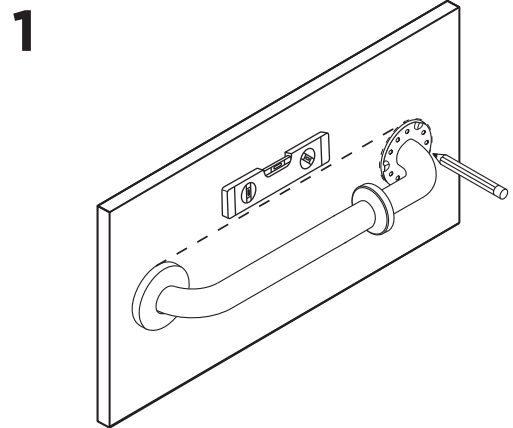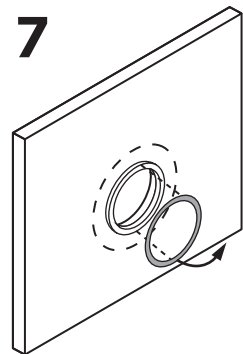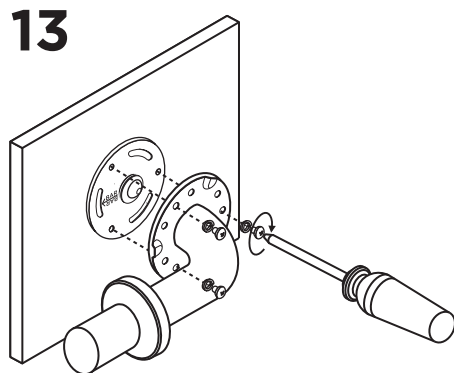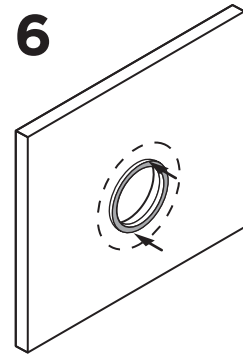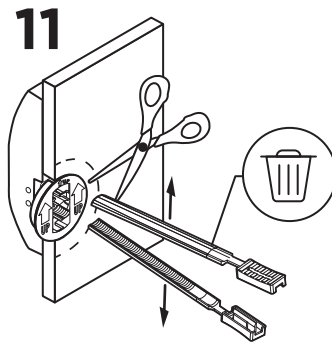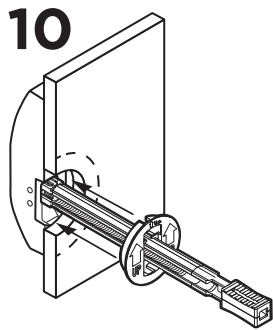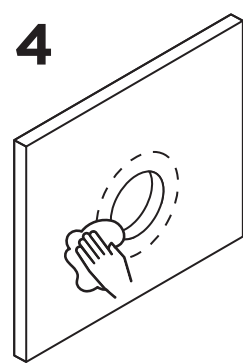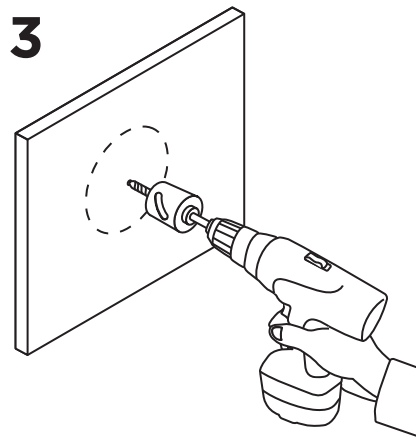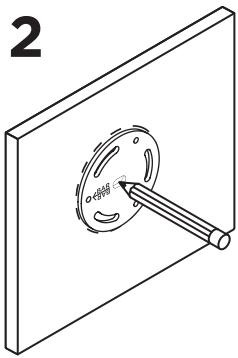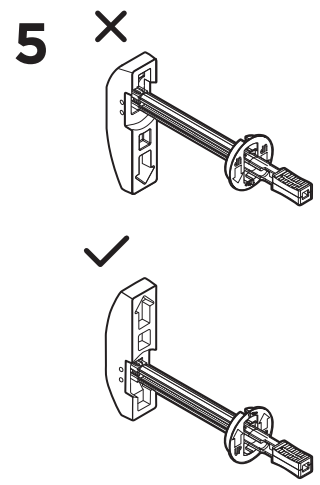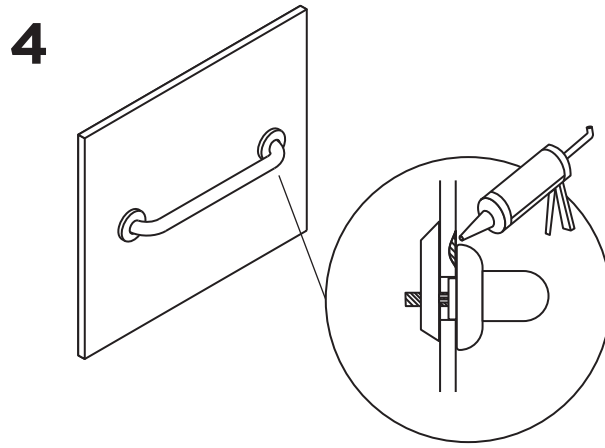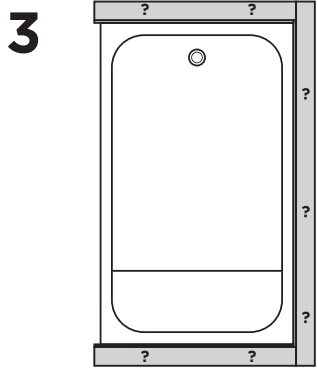Thank you for choosing the HealthCraft Easy Mount Anchor. Please read and understand the instructions in this manual. It is your responsibility to see that your Easy Mount Anchor is properly assembled, installed, and cared for. Failure to follow instructions in this manual could result in serious injury or death. If you are not equipped to undertake the outlined work, we would recommend that you have your Easy Mount Anchor installed by a qualified contractor.

The Easy Mount Anchor is intended to assist in grab bar installations where there is no structural wood backing. The EasyMount Anchor distributes the loads onto wall substrates (min ¼" fiberglass / ½" drywall, max wall thickness 1½") to support the forces applied to a grab bar. It requires a 3½" space behind the wall. It can be used on both flanges of a grab bar, or 1 grab bar flange provided the other flange is secured directly to structural support. The anchor is positioned behind the wall and is held in place by a plastic retainer and ratcheting rails. The stainless steel mounting plate is installed on the front of the wall, and bolts directly to the anchor. The mounting plate provides threaded holes for a grab bar to be secured with machine screws. The foam gasket on the back of the mounting plate provides a seal to protect from water penetration. The maximum user weight for this product is 300lbs / 136kgs. It is not to be used in any other way than described above.

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Products are covered by a limited lifetime warranty against defects in materials and workmanship for the original purchaser. Warranty excludes products that have been damaged through misuse, accidental damage, alteration, normal wear and tear, wood material and stain, or the use of corrosive or abrasive cleaning products.

Buyer hereby indemnifies, agrees to hold harmless and defend HealthCraft Products Inc. from and against any and all liabilities, claims, (founded and unfounded), losses, damages, costs and expenses (including without limitation consequential damages and reasonable professional fees) resulting from buyers specification, application, or improper use of goods described hereon; buyers omission or neglect. HealthCraft Products Inc. does not assume any liability for damage resulting from services performed by others or faulty installation, misuse or misapplication of goods sold by HealthCraft Products Inc. HealthCraft Products Inc. shall not be liable for prospective profits or special, indirect, or consequential damages, or for the cost of any corrective work done without HealthCraft Products Inc. prior approval. HealthCraft Products Inc. total liability hereunder shall in no event exceed the purchase price of the goods specified hereon. Specifications subject to change without notice.

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