

EasyMount™ Anchor

for easy grab bar installation into hollow walls.



Advantages

The EasyMount Anchor allows for quick and easy grab bar installation - exactly where support is needed without adding wood blocking. It is compatible with HealthCraft's EasyMount and PLUS Series grab bars.

Works in:



Fiberglass:

1/8" or 1/4"



Drywall+Tile:

MAX 1 1/2"



Drywall:

1/2" or 5/8"

With the EasyMount Anchor you can:



Support up to 300 lbs




ADA

Make a grab bar ADA Compliant with one or no studs



Waterproof a grab bar installation

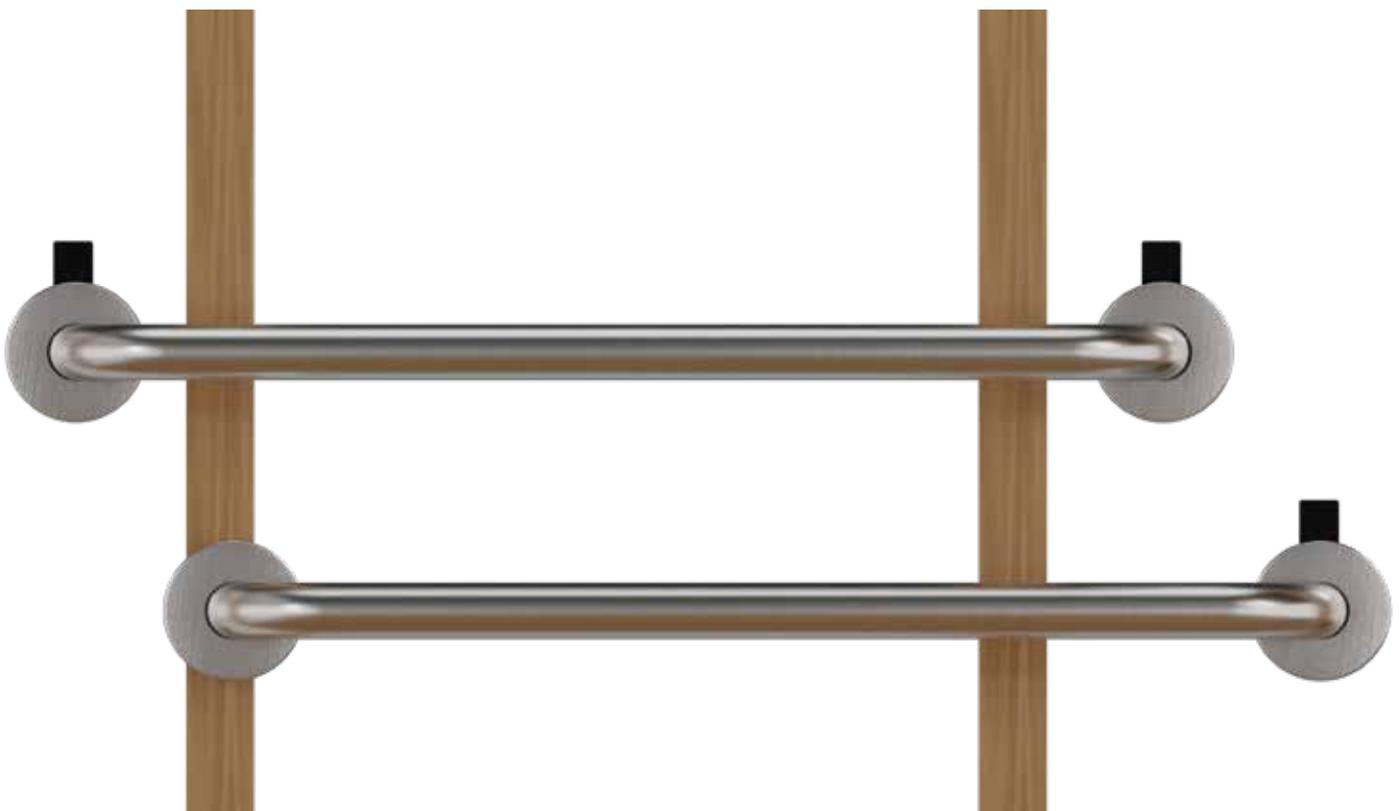

Installation Overview

1



[View Video](#)



Make 1¼" hole

2



Insert anchor

3



Slide retainer

4



Cut rails & discard

5



Install the mounting plate

6



Install grab bar

Specs

Anchor Material	Nylon
Plate Material	#304 Stainless Steel
Hardware Material	#304 Stainless Steel
Weight Capacity	300 lbs  Allows for compliance to ADA 610.4 Structural Strength
Wall hole size	1¼"
Wall clearance required	3½"

Works with these walls substrates and thickness

Fiberglass: ⅛" or ¼"

Drywall: ½" or ⅝"

Drywall + Tile: MAX 1½"

Parts

