



Pullcord Station, Crest Replacement for Jeron® Emergency Response Systems

2332-23002C


Function Table

Jeron Wire Color	Crest Wire Color	Function
Yellow	Yellow	Switch Common
Brown	Brown	Switch Common
White/Green	Red	(+) VDC (To LED)
White/Brown	Grey	Normally Open
Orange	Orange	Normally Open
White/Orange	White	Normally Closed
Black	Black	Normally Closed
Green	Green	Chassis Ground

Typical Installation

1. Replace existing station with replacement, referencing the function table wire for wire.
2. Fasten green lead to the wall box.
3. Always test all nurse call stations for correct operation before placing into service.

Electrical Schematic

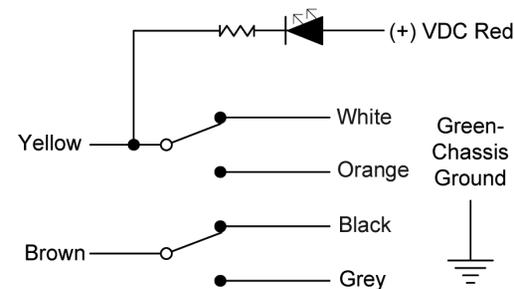

Specifications

ELECTRICAL

Voltage Rating & Current: 24VDC @ 20mA

MECHANICAL

Terminals:

Mounting:

Wire Leads

Fits standard single gang back box

SIZE MEASUREMENTS

Plate:

4.5" x 2.75" (1 Gang)