



410374C



410373C

Patient Stations, Crest Replacement for Faraday® System

P/N 410374C
 410373C

Specifications


| | |
|--------------------------|---|
| ELECTRICAL | |
| Voltage Rating: | 24 VAC input |
| MECHANICAL | |
| Connector: | Screw |
| Wire Termination: | Screw |
| Mounting: | Fits on standard single gang back box |
| SIZE MEASUREMENTS | |
| Plate: | 4.5" x 2.75" (Single Gang) 4.5" x 4.562" (Dual Gang) |

Typical Installation

1. Review designations on the Main Terminal: 1, 2, 3, B and C.
- Helpful hint:** Mark all conductors before removing the station being replaced.
2. Note these designations on the circuit board of Crest Replacement Station.
 3. Screw wires from the Nurse Call terminals to the Crest Replacement Station.
 4. Cap any unused wires.
 5. Always test the nurse call station for correct operation (including cord out) before placing into service.
 6. Extend actuator fully when not in use.

Circuit Board Terminal Descriptions:

| | |
|---|------------------------|
| C | 24 V |
| 1 | To Light (Annunciator) |
| 2 | To Light (Corridor) |
| 3 | To Light (Zone) |
| B | Sounder/Buzzer |

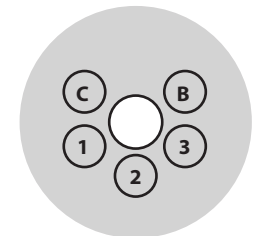

Front of Receptacle