



Crest 3-Position Pullcord Station with Call Placed Light

4303-5

Typical Installation

1. The Yellow, White and Red wires are normally open outputs usually connected to the corridor light, the nurse station light and a remote duty station light.
2. The Blue wire is a normally open, momentary contact which is usually connected to the buzzer/sounder.
3. The Grey and Violet wires are normally closed contacts that have current available when the switch is in the "off" (up) position. Cut off or cap if not used in your system.
4. Orange is the incoming power.
5. Green is used for "Ground".
6. The unconnected Black wire from the nurse light is to be connected to the lamp common.
7. Cut off or cap all unused wires.

Specifications

ELECTRICAL

Current and Voltage Rating: 1.5A @ 28VDC and 0.50A @ 28

MECHANICAL

Terminals: Brass, silver plated
Index Angle: Spring return and positive, 30 degrees
Contact Life: 25,000 cycles minimum
Mounting: Fits standard single gang back box

SIZE MEASUREMENTS

Plate: 4.5" x 2.75" (1 Gang)

Function Table

Color	Orange	Violet	Grey	Red	Yellow	White	Blue
Original Terminal Designation	BC	NC	NC	PC	BL	D	BZ
Up Position	Power	ON	ON	OFF	OFF	OFF	OFF
Middle Position	Power	OFF	OFF	ON	ON	ON	OFF
Down Position (Momentary)	Power	OFF	OFF	ON	ON	ON	ON
Green wire is ground.							

Electrical Schematic

