

Call Crest Technical Support for assistance at: **1-800-328-8908**

www.cresthealthcare.com ©2012 Crest Healthcare Supply



Crest Sentry II[™] Door Alarm

4701, 4702 and 4702L

Typical Installation

- 1. Open unit and place back panel against door frame so that the ball switch is depressed when door is closed.
- 2. Fasten back panel with 2 screws furnished with unit.
- 3. Install 9V battery. Slide door alarm front case down over the alarm back plate. Fasten with screw through bottom lip of the back plate.
- 4. Activate alarm by turning the key.
- 5. For models 4702 & 4702L alarm will sound when the door is opened. Alarm will continue to sound after the door has closed. For model 4701 the alarm will stop after door has closed.
- 6. Silence alarm with key, (except for model 4701).
- 7. Re-activate alarm by turning key on.
- 8. To replace battery, remove screws on side of unit. Slide alarm front case up and off of back plate. The battery will be exposed for easy replacement with a standard 9V battery.

Specifications

ELECTRICAL

Current and Voltage Rating: 9VDC from standard battery

MECHANICAL

Alarm Switch: Switch: Sounder: Circuit:	Momentary roller ball type Key operated on/off switch Solid state piezoelectric type Solid state latching design: 4702 & 4702L Non-Latching design: 4701
MATERIAL:	400 series stainless steel
SIZE:	4.6"H x 2.9"W x 1.5"D
MOUNTING:	Snaps into stainless steel bracket