

CREST

HEALTHCARE SUPPLY®

P.O. Box 727, 195 Third Street
Dassel, Minnesota 55325-0727

For Technical Assistance
Call Toll-Free U.S. & Canada

1-800-328-8908

Technicians On Duty

©2005 Crest Healthcare Supply

B-39073

5343 Series Mushroom Button Emergency Call Stations


(For all Momentary & Latching Models)

INSTALLATION INSTRUCTIONS

CREST

HEALTHCARE SUPPLY®

reliable | responsive | innovative

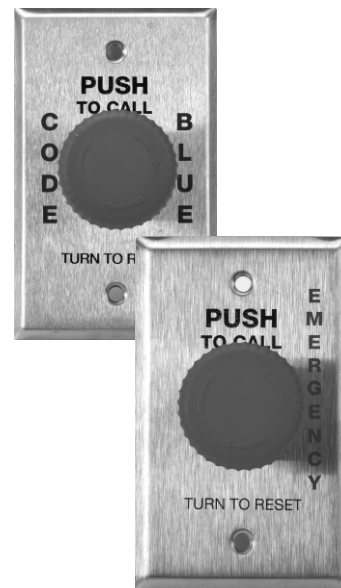

SPECIFICATIONS

ELECTRICAL

Current and voltage rating: 10 Amps, @ 250 Volts AC.

MECHANICAL

Mounting: Fits standard single gang back box.

TYPICAL HOOK-UP

1. The orange and blue wires indicate the normally open contacts.
2. The black and brown wires indicate the normally closed contacts.

Note: Station is available with a maximum of three single pole contacts, in either normally open, normally closed, or combination of normally open and closed switching arrangements.

POSITION / FUNCTION TABLE

| COLOR | Orange | Black | Blue | Brown |
|--------------------|--------|-------|------|-------|
| Normal Position | Power | Power | Off | On |
| Depressed Position | Power | Power | On | Off |

ELECTRICAL SCHEMATIC

Typical wiring for (Normally Open)



Typical wiring for (Normally Closed)



Note: Use 18 Gauge Stranded Wires