



6121C



6123C

Single & Dual ¼" Phone Jack Patient Stations, Crest Replacement for Jeron® Pro-Alert 610

P/N 6121C (Single Stainless Steel Faceplate)
6121CPL (Single Plastic Faceplate)
6123C (Dual Stainless Steel Faceplate)
6123CPL (Dual Plastic Faceplate)

Typical Installation

1. Review wire designations on the old station
2. Replace the old station with Crest replacement station wire for wire (Review the Wire Color/Function Table for reference)
- 3) All unused wires must be capped
- 4) Always test nurse call station for correct operation before placing into service

Specifications

ELECTRICAL	
Voltage Rating:	24 VDC input
MECHANICAL	
Terminals:	Wire Leads and Header Connectors
Mounting:	Fits on standard single/dual gang back box
SIZE MEASUREMENTS	
Plate:	4.5" x 2.75"

Wire Color/Function Table

Location	Crest Wire Color	Function
P1	Black	Power
	White	GND
	Red	EC
	Blue	A
	Orange	CO
	Yellow	RST

Schematic

