

## Single & Dual 8-pin DIN Patient Stations, Crest Replacement for Jeron®

P/N 6420C  
6422C

### Typical Installation

- 1) Review header connector and wire designations on the old station.
- 2) Replace the old station with the Crest replacement station while referring to position "1" for proper orientation of wires/headers.
- 3) Review the Function Table for reference if needed.
- 4) All unused wires must be capped.
- 5) Always test nurse call station for correct operation before placing into service.

### Specifications

**ELECTRICAL**

**Voltage Rating and Current:**

28 VDC input

**MECHANICAL**

**Terminals:**

Wire Lead and Header Connectors

**Mounting:**

Fits on standard dual gang back box

**SIZE MEASUREMENTS**

**Plate:**

4.5" x 4.562" (dual gang)



### Function Table

Location	Function
P1	P G S FC WD L C RST CC CO
P2	SPK+ SPK-
P3	AC1 AC2
P4 and P5	LSP LTV LCN LAX