



7603EC



7613EC

Patient Stations, Crest Replacement for Edwards® System

P/N 7603EC
 7613EC

Typical Installation

1. Review designations on the Main Terminal: 1, 2, 3, B and C.
- Helpful hint:** Mark all conductors before removing the station being replaced.
2. Note these designations on the circuit board of Crest Replacement Station.
3. Screw wires from the Nurse Call terminals to the Crest Replacement Station.
4. Cap any unused wires.
5. Always test the nurse call station for correct operation (including cord out) before placing into service.

Specifications

ELECTRICAL	
Voltage Rating:	24 VAC input
MECHANICAL	
Connector:	Screw
Wire Termination:	Screw
Mounting:	Fits on standard single gang back box
SIZE MEASUREMENTS	
Plate:	4.5" x 2.75" (Single Gang) 4.5" x 4.562" (Dual Gang)

Circuit Board Terminal Descriptions:

T1			T2	
1	2	3	C	B
1, 2, 3			Lamp Circuits	
B			Buzzer	
C			HOT (24 VAC)	
7613EC Wire Terminal White & Black			Station Nurse Lamp	

Schematic

