



8721C



8723C



8710C

Bedside Station, Crest Replacement for Jeron® EC-300

P/N 8721C 8721CPL
8723C 8723CPL
8710C

Typical Installation

- 1) Replace existing station with Crest replacement station, wire for wire (Review the Function Table for reference)
- 2) Fasten eyelet of green lead to the backside of the faceplate through the mounting hole when screwing station to wall box
- 3) Always test all nurse call stations for correct operation (including cord out for 8721C, 8721CPL, 8723C and 8723CPL) before placing into service

Specifications

ELECTRICAL

Voltage Rating & Current 24 VDC input @ 20 mA

MECHANICAL

Terminals: Wire Leads

Mounting: Fits standard single gang back box

SIZE MEASUREMENTS

Plate: 4.5" x 2.75" (1-Gang)

Function Table

Jeron Original Wire Color	Function
White/Green	(A) Output
White/Orange	(B) Output
Brown	(-) 24 VDC
Orange	(+) 24 VDC
Green	Ground