

Pushbutton Station, Crest Replacement for Cornell® Systems



B-111C (stainless 1-gang), B-111CPL (plastic 1-gang),
 B-113C (stainless 1-gang), B-113CPL (plastic 1-gang),
 B-122C (stainless 1-gang), B-122CPL (plastic 1-gang),
 B-222C (stainless 2-gang), B-222CPL (plastic 2-gang)
 B-311C (3-gang), B-322C (3-gang)

Technical Specifications

Current & Voltage Rating:	24 VDC input
Connector:	Terminal Block
Wire Termination:	Screw
Mounting:	Fits standard gang back box. Refer to size measurements.
Plate:	4.5" x 2.75" (1-gang) 4.5" x 4.562" (2-gang) 4.5" x 6.375" (3-gang)



B-111C



B-113C

Typical Installation

1. Review wire designations on old station: CS, CS, -, +, E, A, GND
2. Note these designations on the wire terminal of Crest replacement station (see Function Table for description)
3. Always test all nurse call stations for correct operation (including cord out if applicable) before placing into service

Wire Color Comparison Table

Position on PCB	Label on PCB	Function/Description
1	CS	Cancel Switch
2	CS	Cancel Switch
3	(-)	24VDC Ground
4	(+)	24VDC Input
5	E	Extra Light
6	A	Annuciator
7	GND	Ground Terminal for Faceplate

Crest Technical Support: **1-800-328-8908**

Specifications subject to change without notice.

Visit CrestHealthcare.com for additional technical documents.