



Single Sideguard 7-Pin Station, Crest Replacement for Executone®

P/N CCPCS-W43C

Specifications

ELECTRICAL	
Voltage Rating:	24 VDC input
MECHANICAL	
Terminals:	Edge connectors
Mounting:	Fits on standard triple gang back box
SIZE MEASUREMENTS	
Plate:	6.375" x 4.5" (triple gang)

Description

Single Sideguard station comes with 2-way audio communication feature. Uses 7-pin multi – prong receptacle for connection to pillow speaker or call cord that can be used for call placement. Green Call indicator and red Cancel indicator are provided for visual confirmation.

Typical Installation

1. Requires three 9-pin edge connectors (not included, call for part number if needed).
2. Review wire designations on the old station.
3. Replace the old station with Crest replacement station wire for wire.
4. All unused wires must be capped.
5. Always test nurse call station for correct operation before placing into service.