



Patient Stations, Crest Replacement for Cornell® E-101, E-105 & E-108

P/N E-101C (Replacement for E-101 & E-105)
 E-108C (Replacement for E-108)

Product Description

Crest Replacement for Cornell Emergency Stations are mounted on a 1-gang stainless steel faceplate with proper designations. Features a push-for-help actuation where single pushbutton initiates a call and an additional push will cancel the call. Can be used as emergency patient or staff stations.

Typical Installation

1. Review PCB designations on the old station.
2. Replace the old station with Crest replacement station wire for wire (Review the Function Table for reference.)
3. All unused wires must be capped.
4. Always test nurse call station for correct operation before placing into service.

Specifications

ELECTRICAL	
Voltage Rating:	24 VDC input
MECHANICAL	
Terminals:	Terminal Block
Mounting:	Fits on standard single gang back box
SIZE MEASUREMENTS	
Plate:	4.5" x 2.75"

Function Table

PCB Designator	Function
RL	Light (Not present in E-108C)
+	+24 VDC Input
L	Looping Input
A	Annunciator
E	Extra (Same as A)
-	Negative Input
G	Chassis Ground

Schematic

