



Call Crest Technical Support for  
assistance at: **1-800-328-8908**

[www.cresthealthcare.com](http://www.cresthealthcare.com)

©2012 Crest Healthcare Supply

B-39176  
09-04-12  
page 1 of 1



## Emergency Pullcord Stations, Crest Replacement for Cornell® Systems

E-114C  
E-144C

### Function Table

Cornell Wire Color	Crest Wire Color
Brown	Brown
Grey	Grey
Black	Black
Green	Green

### Typical Installation

1. Replace existing station with Crest replacement wire for wire. Note the Wire Color Comparison Table for reference.
2. Fasten eyelet on green wire to the backside of the faceplate with mounting screw when screwing the station to the wall box.
3. Always test nurse call station for correct operation before placing into service.

### Specifications

#### ELECTRICAL

**Voltage Rating:** 24VDC input

#### MECHANICAL

**Terminals:** Wire Leads

**Mounting:** Fits standard single gang back box

#### SIZE MEASUREMENTS

**Plate:** 4.5" x 2.75" (1 Gang)

### Electrical Schematic

