




HSS-103-CC



HSS-103-CC-CC

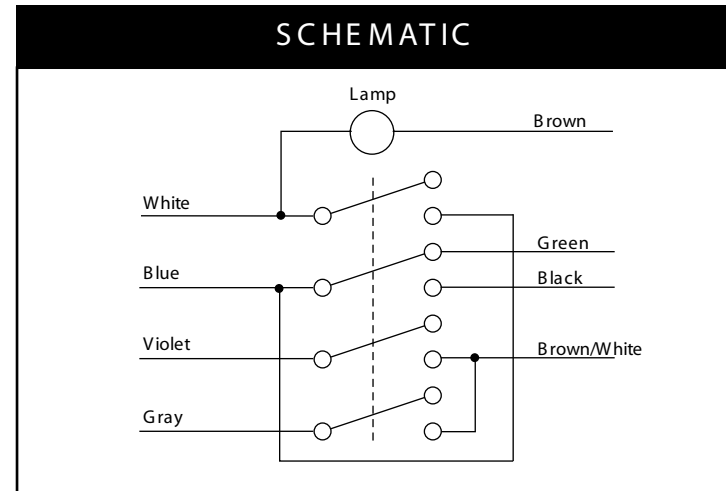

## Emergency Stations

P/N HSS-103-CC  
 HSS-103-CC-CC

## Wire Color Comparison Table

Ektacom	Crest
Red	Brown
Orange	Green
Green	Blue
Grey	Violet
Blue	Grey
Yellow	White
Brown	Black
White	Brown/White

## Typical Installation

1. Replace existing station with Crest replacement, referencing the wire color comparison table on right.
2. When installing ensure the o-rings and gasket are correctly used to ensure a waterproof seal.
3. Always test all nurse call stations for correct operation (including cord out if applicable) before placing into service.