

Dual Call Cord Station, Crest Replacement for Rauland® Responder® IV

NC2JACKC (Stainless Steel Faceplate)

NC2JACKCPL (Plastic Faceplate)

CREST
HEALTHCARE®



NC2JACKC

Technical Specifications

Voltage Rating:	14 VDC @ 100 mA
Terminals:	S-Bus Jack
Mounting:	Fits on standard single gang back box
Plate:	4.5" x 2.75" (Single Gang)

Typical Installation

1. Configure the replacement station as either STATION A or STATION B by placing the station select jumper on the necessary pins of header JU1
2. Configure the replacement station CALL 1 as either MOMENTARY or LATCHED by placing the select jumper on the necessary pins of header JU3
3. Configure the replacement station CALL 2 as either MOMENTARY or LATCHED by placing the select jumper on the necessary pins of header JU2
4. Replace existing station with Crest replacement by connecting the six-pin modular plug to the station's modular S-Bus jack
5. Always test all nurse call stations for correct operation before placing into service

Crest Technical Support: 1-800-328-8908

Specifications subject to change without notice.

Visit [CrestHealthcare.com](https://www.CrestHealthcare.com) for additional technical documents.