

Pushbutton/Pullcord Station, Crest Replacement for Rauland® Responder® IV NCPBPHC & NCPCS1C



NCPBPHC



NCPCS1C (waterproof)

Technical Specifications

Voltage Rating:	14 VDC @ 100 mA
Terminals:	S-Bus Jack, Remote Cancel Switch Connector
Mounting:	Fits on standard single gang back box
Plate:	4.5" x 2.75" (Single Gang)

Typical Installation

1. Configure the replacement station as either station A or station B by using the station select switch on the back of the station.
2. Replace existing station with Crest replacement by connecting the six-pin modular plug to the station's modular S-Bus jack.
3. If using the optional remote cancel switch, connect the remote cancel switch cable to header J3 on the Crest replacement station.
4. When installing the waterproof pullcord station, ensure the O-rings and gasket are correctly used to provide a waterproof seal.
5. Always test all nurse call stations for correct operation before placing into service.

Crest Technical Support: 1-800-328-8908

Specifications subject to change without notice.
Visit CrestHealthcare.com for additional technical documents.