


## TXP-11(V2) User Instructions

V0.1

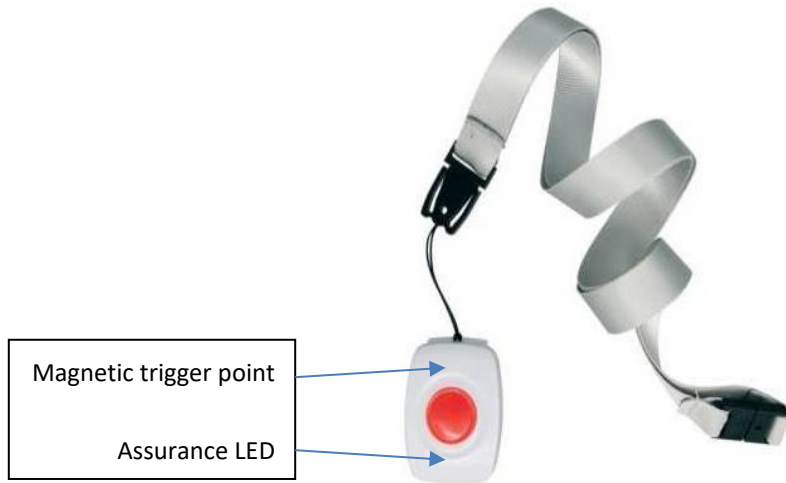

### Behavior Chart

Call Triggered	LED stays ON for 15s
Man-down Detected	LED blinks every second for 5s - Call activated if no further motion - Call cancelled if motion resumes
Call Reset	LED flashes 3x
Low Battery	LED blinks once every 5s

### Compatibility

Room Indicators	ILB-21 Over Door Light (LoRa)
Call Systems	ZP-01(S) Network Repeater
Pagers	PAG-11 Caregiver Alert Pager PAG-31 Economy Pager
Bed Monitors	NGM-21 CordFree Monitor NGM-41 Economy Cordless
Door Monitor	DMS-02 Door Strip Sensor
Nurse Call Interface	CMEX-11 Fixed ¼" Jack CMEX-21 Removable Fly Cable
Accessories	WOL-01 Faux leather watch strap WB-01 Unbreakable strap CR2450 Lithium coin battery CK-01 Caregiver Key magnet

### Programming


Enter programming by holding a magnet to the trigger point. When the LED begins flashing, hold the button until the LED stays ON, then release the button and remove the magnet. To enter a digit, press the button that number of times, then touch the magnet – the LED will blink to indicate the digit has been entered. Each program code consists of two digits.

<b>Settings</b>	
Setting 1 – Call & Reset	Enter code 10 - DEFAULT
Setting 2 – Call, Reset & Man-down	Enter code 11
Setting 3 – Call, Reset & Door monitor	Enter code 12
Setting 4- Call, Reset, Door monitor & Man-down	Enter code 13
Setting 5 – Door monitor only	Enter code 14
<b>Latching</b>	
Turn latching OFF	Enter code 20 - DEFAULT
Turn latching ON	Enter code 21
<b>GSM Dialer</b>	
Compatibility OFF	Enter code 30 - DEFAULT
Compatibility ON	Enter code 31
<b>Tracking</b>	
Turn tracking OFF	Enter code 40 - DEFAULT
Turn tracking ON	Enter code 41
<b>Sensitivity</b>	
Low sensitivity	Enter code 50 - DEFAULT
Med sensitivity	Enter code 51
High sensitivity	Enter code 52

**WARNING:** Risk of explosion if battery is replaced by an incorrect type. Replace only with the same or equivalent type.

**CAUTION:** Lithium batteries cannot be disposed of with normal household waste. For disposal or recycling instructions please contact your local authorities.

**Testing the Man-down**

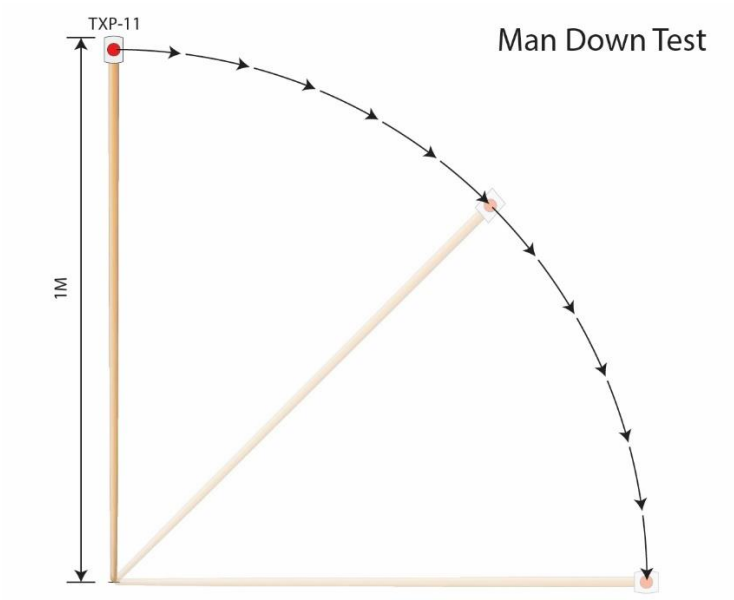

Low  
Med  
High