


## WCP-11C User Instructions

V0.1

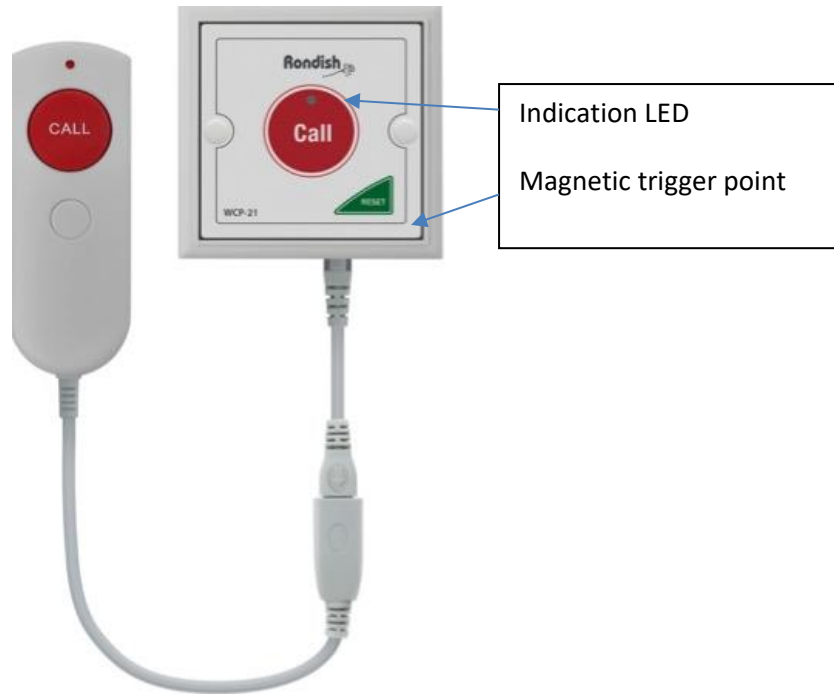

### Activate/Deactivate Tracking

- Apply magnet until the LED begins flashing, then press and hold Reset
- The LED will flash rapidly to indicate tracking is enabled
- The LED will be steady on to indicate tracking is disabled

### Call Trigger

- Press the Call button
- Double-pressing Call will activate an Emergency call
- Press Reset to cancel the call

**WARNING:** Risk of explosion if battery is replaced by an incorrect type. Replace only with the same or equivalent type.

**CAUTION:** Lithium batteries cannot be disposed of with normal household waste. For disposal or recycling instructions please contact your local authorities.

## Programming

- Enter programming by applying a magnet to the trigger point and holding Call the indication LED will stay ON when it is ready. When you have completed the input string the WCP-11C will automatically exit programming
- Enter code 28 to change program settings
- Key in 13 digits: t T1T2 D1D2D3D4D5D6 P LLL , where
- t = 1
- T1T2 - Alarm type
  - Alarm - 06 (default)
  - Emergency - 07
- D1...D6 - 6-digit Device ID
- P - Product code
  - Toilet button - 3
  - Bedhead unit - 8 (default)
- LLL - Area code

## Behavior Chart

|   |                             |
|---|-----------------------------|
| Call Triggered - includes Call button and handset | Beeps, LED stays ON for 15s |
| Emergency Triggered                               | Beeps, LED flashes for 15s  |
| Call Reset  | Beeps, LED flashes 3x       |
| Low Battery                                       | LED blinks once every 5s    |

## Compatibility

|                      |  |
|----------------------|--|
| Room Indicators      | ILB-21 Over Door Light (LoRa)  |
| Call Systems         | ZP-01(S) Network Repeater<br>DON-20 Receiver Dongle                                |
| Pagers               | PAG-11 Caregiver Alert Pager<br>PAG-31 Economy Pager                               |
| Nurse Call Interface | CMEX-11 Fixed ¼" Jack<br>CMEX-21 Removable Fly Cable                               |
| Accessories          | NEC-12 Nurse Call Handset<br>MB-01 Single-Gang Bracket<br>CR2477 Lithium Batteries |